# normomar<sup>®</sup>

spray

## Agua Marina ISOTÓNICA

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HIGIENE NASAL DIARIA

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P1.11

# normomar<sup>®</sup> spray

## Sea Water ISOTONIC Aloe Vera and Hyaluronic Acid DAILY NASAL HYGIENE

### **NORMOMAR<sup>®</sup> SEA WATER ISOTONIC WITH ALOE VERA AND HYALURONIC ACID NASAL SPRAY**

A proper nasal hygiene is important for adults, children and babies. The excess of mucus in the nasal cavities may increase the microbial proliferation and the excessive dryness of the nasal mucosa.

**NORMOMAR<sup>®</sup> Sea Water Isotonic with Aloe Vera and Hyaluronic Acid** hydrates nasal cavities thanks to its mechanical action and helps washing and extracting the mucus. The condition of the nasal mucosa is essential in breathing as it filters, humidifies and warms the air we breathe. There are certain circumstances that alter the nasal mucosa: rhinitis, allergies, dry nostril after surgical processes, air conditioners and drug treatments.

Thanks to Aloe Vera, which easily allows the permeability of the nasal mucosa, and 0.2% Hyaluronic Acid, whose high moisturizing power helps to restore nasal mucosa and promote its normal operation, this product provides long-lasting hydration and a soothing effect.

**NORMOMAR<sup>®</sup> Sea Water Isotonic with Aloe Vera and Hyaluronic Acid** maintains the specific properties of natural sea water.

The sea water comes from the Cancale Bay, in Brittany, which is renowned for the top quality of its water. The Cancale Bay boasts an exceptional tidal range (12 m) and is swept by strong prevailing currents preventing pollution of the waters. This unique combination ensures that the water is constantly self-cleaning and is, therefore, consistently pure. Sampling is performed automatically when the water quality is optimal (after the high tide period). The sampling area is classified as Class A, and is subject to continuous monitoring and sanitary controls (microbiological, chemical contamination, phytoplankton, pH...).

#### **PRESENTATION:**

Pressurized spray bottle of 120 ml with nasal applicator in force 1.

#### **COMPOSITION:**

Purified water, sea water (0.9% isotonic solution), Aloe Vera (0.5%) and sodium hyaluronate (0.2%).

#### **INDICATIONS:**

A proper use of **NORMOMAR<sup>®</sup> Sea Water Isotonic with Aloe Vera and Hyaluronic Acid** in the daily nasal hygiene of babies from 1 year, children and adults

allows offering therapeutic aid in some basic indications such as: nasopharyngitis, obstructive rhinitis and sinusitis, non-productive dry nose, hoarseness due to dry mucous membranes, and it performs as adjuvant in ENT and it serves as an adjuvant in ENT pediatric treatments.

#### **INSTRUCTIONS FOR USE:**

1. Gently insert the tip at the entrance of the nostril.
2. Press the spray head.
3. Allow to drain the excess solution in washing to extract the mucus, and then blow.
4. Repeat on the other nostril.
5. Clean the tip with water and dry.
6. Cover the nasal applicator with the protective cap until the next use.

Apply in each nostril as often as necessary.

#### **INSTRUCTIONS TO APPLY THE PRODUCT TO THE BABY:**

1. Lay your baby down with the head slightly tilted to one side to facilitate the entry of sea water in the nostril.
2. Pulverize for a few seconds pressing the spray head.
3. Sit up the baby to let the nose drain out. Collect the mucus on a tissue.
4. Repeat on the other nostril.
5. Gently wipe around the nostrils without putting the tissue into the baby's nose. Clean the tip with water and dry.

Apply to each nostril 2 to 3 times daily.

#### **Warnings:**

- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- Keep out of the reach of children.
- Do not use the product after the expiry date which is stated on the box and the bottle.
- Store in a cool, dry place. Store below 40° C and protected from direct sunlight or other heat sources.
- In case of a known or suspected allergy to any of the ingredients of this medical device, it is advisable not to use it.

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LABORATORIOS NORMON, S.A.